

## Equine Fecal Egg Count Submission Form

RANCH/FARM: \_\_\_\_\_

OWNER: \_\_\_\_\_

ADDRESS: \_\_\_\_\_

PHONE: \_\_\_\_\_

FAX: \_\_\_\_\_

EMAIL: \_\_\_\_\_

**Report Method:**

<input type="checkbox"/> Fax	<input type="checkbox"/> Phone	<input type="checkbox"/> Email	<input type="checkbox"/> Mail
------------------------------	--------------------------------	--------------------------------	-------------------------------

EQUINE ID: \_\_\_\_\_

DATE COLLECTED: \_\_\_\_\_

Deworming Product(s) Used and Date: \_\_\_\_\_

\_\_\_\_\_

**\*PLEASE SEE INSTRUCTIONS ON REVERSE SIDE\***

**FOR LAB USE ONLY:**

DATE RECEIVED: \_\_\_\_\_

SAMPLE CONDITION: \_\_\_\_\_

SIGNATURE:

\_\_\_\_\_

## Instructions For Equine FEC Kit

1. Your FEC kit contains the following:
  - Resealable collection bag
  - Nitrile glove to aid in feces collection
  - Pre-addressed and posted bubble mailer for returning sample
  - Cold pack(s) for return shipping
  - Sample submission/instruction sheet
2. Completely Fill Out Sample Submission Information on front side of the instruction sheet prior to collecting sample.
3. Freeze the cold pack(s) completely. Cool the fecal sample prior to packing but **DO NOT FREEZE THE SAMPLE.**
4. Label the resealable collection bag with permanent marker prior to sample collection.
5. Collect a fresh fecal sample about the size of a large chicken egg soon after the horse has passed it. One individual sample per bag.
6. Be sure the sample bag is securely sealed. Place sample bag, frozen cold pack(s) and completed sample submission form in bubble mailer and mail to Precision Diagnostics LLC.



245 White Oak Rd

New Holland, PA 17517

(717)354-2178    [www.precisiond.com](http://www.precisiond.com)    [pdlab@precisiond.com](mailto:pdlab@precisiond.com)